

Sensitive?

Not anymore

UltraEZ™ Desensitizing Gel

- Provides fast, easy treatment for sensitive smiles
- Prefilled UltraFit™ tray adapts to any smile for a comfortable fit
- Also available in syringes for use with custom trays



Additional features of UltraEZ desensitizing gel

- Quickly treats sensitivity caused by tooth whitening, abrasion, root exposure, and thermal or chemical changes
- Can be used before and after whitening treatments to dramatically reduce sensitivity

Instructions



1. Remove product from packaging. "U" - Upper tray "L" - Lower tray



Before wearing UltraFit tray in the mouth



UltraFit tray after just 10 minutes in the mouth



2. Position upper tray on teeth.



3. Bite firmly, then suck on tray for 2 seconds.



4. Remove colored outer tray, leaving the white inner tray on teeth. Repeat the process for the lower tray.



5. Wear for 15 minutes to 1 hour per day or as directed by your dentist. After indicated wear time, remove trays and brush teeth.

1008 - UltraEZ Syringe 4pk
1007 - UltraEZ Econo Syringe 20pk
1.2 ml syringes



5743 - UltraEZ Mini Tray Combo 4pk
4 x Each upper/lower trays



Featuring the UltraFit™ tray

5721 - UltraEZ Tray Combo 10pk
10 x Each upper/lower trays



Featuring the UltraFit™ tray