

Quad Matrix System™

NEW!

The ideal addition to your sectional matrix system for complex cases.

Confidently address challenging restorations including:

Back-to-Back Restorations: Simultaneously restore both sides of adjacent Class IIs with confidence. Two matrix bands, one Quad Wedge, and one Quad Ring make for a streamlined workflow on this common procedure.

Deeper Caries: Use a subgingival Firm Band along with a Quad Wedge for a deeper more secure seal enhanced by the Quad Ring driver tip.

Complex Multi-Tooth Restorations: Use combinations of Universal (blue) and Tall (orange) Quad rings to produce ideal contours and contacts on the most challenging cases. We'll show you how!



Advanced design for complex cases.

Long lasting rings – flash-free results

Drawn-wire nickel titanium produces lasting separating pressure plus ultraadaptive Soft-Face™ tips for reduced flash.

Enhanced cervical seal

The split-tip Quad wedges combined with the driver tip on the Quad rings produce an independent cervical seal on both sides of back-to-back restorations.

Quadrant workflow

Asymmetrical tip design improves ring fit, matrix stabilization and tooth separation all of which are key for achieving optimal results on challenging multiple tooth and back-to-back Class II preparations.




Techniques for success

See Quad cases, technique videos, and frequently asked questions by following this link.

Go beyond basic Class IIs with Quad.

NEW!

While it's great for everyday standard Class IIs, the Quad Matrix System is named for its unique ability to streamline more complex cases. Faced with an entire quadrant of Class IIs like this, you need unique material designs and proven techniques to deliver efficient results you can be proud of.

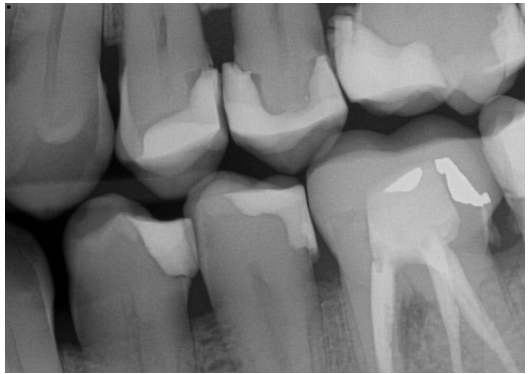

When a case like this sits down in your chair, you need Quad.

Quad Matrix System™

Quad - the ideal addition for complex cases



The Quad kit

Q-KHF-10 Intro Kit:

- 2x Quad Rings (1x each Blue orientation)
- 50x Dead-soft 3D Fusion bands, in 5 sizes
(20x grey 4,4mm; purple 5,6mm; green 6,6mm;
and 5x red 6,0mm; blue 8,7mm),
- 50x Firm bands, in 5 sizes
(10x premolar 4,3mm; small-cervical 5,9mm;
premolar/small molars 5,3mm; molar 6,1mm;
molar/large cervical 7,7mm),
- 80x Quad wedges, in 4 sizes
(20x small/blue, medium/orange, large/green,
anatomical/purple),
- 1x Ring placement forceps FXP01 (page 43),
- 10x VariStrip™ (page 31),
- 5x Margin Elevation Matrix Bands (page 38-39)

Part No.	Description
Q-KHF-10	Intro Kit

All-new asymmetrical **Quad Ring design** with specialized 'driver-tip™' maximizes Quad Wedge adaptation and cervical seal.

NEW!



- Adapts the split-tip Quad wedge to both sides of the embrasure simultaneously
- Seals both cervical margins
- Drives deeper for improved matrix band adaptation and tooth separation
- Soft-Face™ design for superior flash reduction!


Maximum adaptation

The right and left driver tips of the Quad system allow you to select the ideal ring for each quadrant and for each restorative challenge.




○ – Driver-Tip™

Part No.	Description	Stk.
Q400	Quad Universal Rings blue (1x each orientation)	2



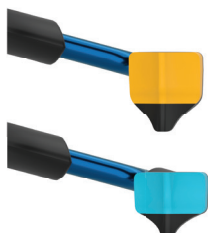


Tall rings (orange) with both left and right-side driver tips are available for simplified stacking.

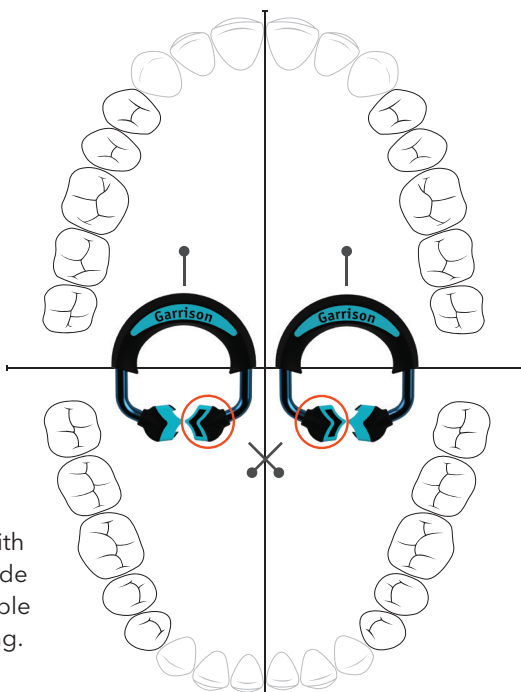

The **Quad Wedge** utilizes a revolutionary split-tip design.

NEW!

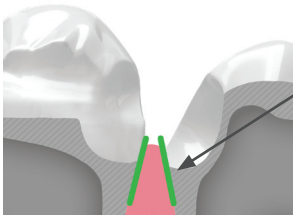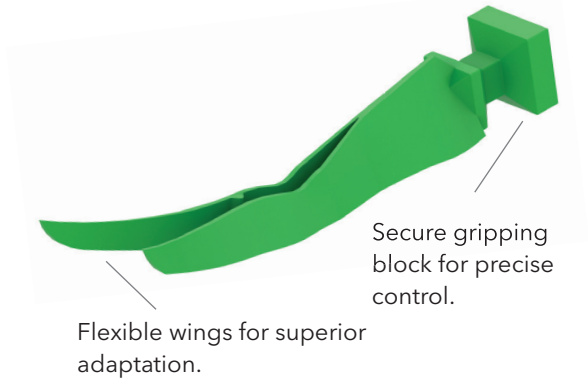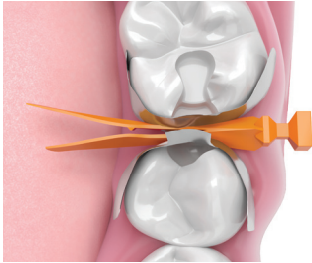

Split tip easily adapts bands to both cervical margins!

Gently slides alongside papilla and seals deeper for a more natural emergence profile— even with mismatched depths.



Seal those concavities!

Quad Wedges include the highly specialized anatomical purple wedge designed for an improved seal when presented with a root concavity.

Three sizes plus the special, anatomically-shaped purple!

- Prevents flash and contamination.
- Improves matrix adaptation.



Kit	Description	Qty.
QK4-M	Quad Wedge Kit (50x small/blue, medium/orange, large/green, anatomical/purple)	200