


# InFocus



## Sterify Gel

Supporting Tissue Healing within the Targeted Healing Approach.



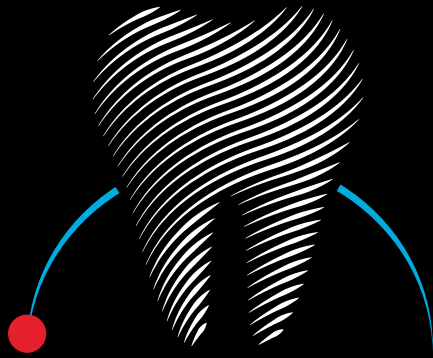

# Targeted Healing Approach

## The Evolution of Individualized Periodontal Care


To meet the unique needs of each patient and clinician in periodontal care, HuFriedyGroup promotes an individualised approach that is implemented through three main phases.

Each phase builds upon the previous one to ensure a tailored, effective, and positive treatment experience for patients, while also supporting the efficiency of dental professionals.



Discover  
more

### ASSESS

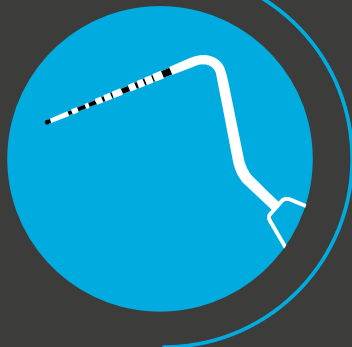

Profiling-driven  
diagnostic for tailored  
intervention

### CARE



Ensuring an optimally  
targeted non-surgical  
treatment choice

### BOOST



Leveraging the  
physiological healing  
capacity


## Complete the Procedure, Maximize Savings

Enjoy **5% extra off** when buying instruments for 2 phases, or **10% extra off** for all 3 phases!

### ASSESS

Profiling-driven diagnostic for tailored intervention

ASSESS

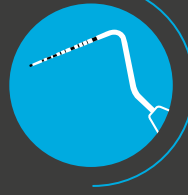

CARE



BOOST



# EFP Probe

## Putting Classification and Guidelines into Practice.

The **EFP Probe** facilitates rapid screening and detailed periodontal assessment. Enabling clinicians to seamlessly implement **state-of-the-art periodontal assessment and treatment decision making** into clinical practice. It can be used in all steps of **periodontal and peri-implant therapy**.

**A special band between 4-6 mm** highlighting the key 'critical probing depths' for non-surgical and surgical interventions

**Detailed markings in 1 mm increments** for patients with more severe disease

**-15%**

**Ball-point tip** makes it easier to enter the periodontal pocket without damaging the sulcus and allows to check for plaque-retentive factors

**A special marking at 3 mm** to indicate the absence of pockets

**The calibrated markings**, based on the 2018 EFP/AAP classification and EFP S3-level guidelines, enhance clarity, consistency, and confidence in clinical decision-making



**EFP Probe, single ended, #6 handle PCPS36**

42,00 €  
**35,70 €**



Discover more



Developed in partnership with the European Federation of Periodontology


## Complete the Procedure, Maximize Savings

Enjoy **5% extra off** when buying instruments for 2 phases, or **10% extra off** for all 3 phases!

### ASSESS

Profiling-driven diagnostic for tailored intervention

ASSESS

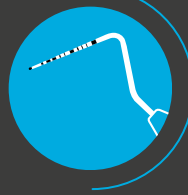

CARE



BOOST



# HD Black Line Mirrors

See your patients, not your instruments.

Glare interferes with visibility. Reducing this disruption to your visibility allows you to see more detail and offer more accurate patient care.

**HD Black Line Mirrors are engineered to optimize clinical outcomes by delivering superior visibility throughout any dental procedure.** The reduced glare properties of the DLC coating and clarity of the HD mirror glass facilitates quicker and more accurate visibility intraorally.



#5 HD Black Line Single-Sided Mirror European Simple Stem  
**MIR5HDBE**

23,40 €  
**19,90 €**

#5 HD Black Line Single-Sided Mirror Cone Socket  
**MIR5HDB**

23,40 €  
**19,90 €**



#5 HD Black Line Double-Sided Mirror EU Simple Stem  
**MIR5DSHDBE**

30,20 €  
**25,70 €**

**-15%**



Download the Brochure

#6 Black Line European Simple Stem Mirror Handle  
**MHE6B**

31,50 €  
**26,80 €**

#6 Black Line Cone Socket Mirror Handle  
**MH6B**

31,50 €  
**26,80 €**


## Complete the Procedure, Maximize Savings

Enjoy **5% extra off** when buying instruments for 2 phases, or **10% extra off** for all 3 phases!

### CARE

Ensuring an optimally targeted non-surgical treatment choice

ASSESS

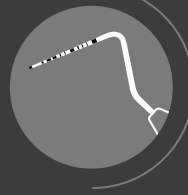

CARE



BOOST



# Harmony™ Ergonomic Scalers and Curettes

Every Patient.  
Every Procedure.  
Every Day.

Using an enhanced finishing process, HuFriedyGroup's professional artisans are creating edges that are over 60 % sharper than the next closest competitor.<sup>4</sup>

Less force means easier, more comfortable scaling for the clinician, which enables them to deliver the best clinical outcome every time.

The advanced sensor based TrueFit™ Technology has measured 2,878,320 data points for pinch force in the finger and pressure applied to the tooth when scaling.<sup>4</sup>



Discover  
more

**Not Too Heavy or Too Light**  
The Harmony™ Ergonomic Scaler and Curette's perfectly balanced handle is similar in weight to our best in class EverEdge™ 2.0 #9 Metal Handle

**Reduces pinch force up to 65%<sup>4</sup>**  
This May alleviate hand fatigue and injury risk due to repetitive motions<sup>5,6,7,8</sup>

**-15%**

**Over 60 % sharper** than the next closest competitor.<sup>1</sup> Using an enhanced finishing process, HuFriedyGroup's professional artisans are creating edges that are over 60% sharper. Less force means easier, more comfortable scaling for the clinician, which enables them to deliver the best clinical outcome every time

**Reduces Pressure on Tooth by 37%<sup>4</sup>**  
Harmony™ Ergonomic Scalers and Curettes May increase patient comfort, while reducing clinician fatigue because it requires less pressure to the tooth when scaling

1) Products tested: Hu-Friedy™ EverEdge™ 2.0 H7 Scaler, American Eagle XP H6-7, PDT H6-7, LM EM/EN H6-7, and GM Hartzell H6-7. For more information, contact David Platt, Hu-Friedy Mfg. Co., LLC, 3232 N. Rockwell St, Chicago, IL 60618. 2) Data on file. Available upon request. 3) Compared to the Hu-Friedy #9 metal handle, these are the nominal values. Data on file. Available on request. 4) Data on file. Available upon request. 5) Int J Dent Hygiene 7, 2009; 159-165 DOI: 10.1111/j.1601-5037.2009.00395.x. Hayes MJ, Cockrell D, Smith DR. A systematic review of musculoskeletal disorders among dental professionals. 6) Rempel, David, et al. "The Effects of Periodontal Curette Handle Weight and Diameter on Arm Pain." The Journal of the American Dental Association, vol. 143, no. 10, 2012, pp. 1105-1113, doi:10.14219/jada.archive.2012.0041. 7) Lalumandier, James A, and Scott D McPhee. "Prevalence and Risk Factors of Hand Problems and Carpal Tunnel Syndrome among Dental Hygienists." Journal of Dental Hygiene, vol. 75, no. 11, 2001, pp. 130-134. 8) Mulimani P, Hoe VCW, Hayes MJ, Idiculla JJ, Abas ABL, Karanth L. Ergonomic interventions for preventing musculoskeletal disorders in dental care practitioners. Cochrane Database of Systematic Reviews 2018, Issue 10. Art. No.: CD011261. DOI: 10.1002/14651858.CD011261.pub2. 9) See www.HuFriedyGroup.eu/Harmony-Scaler for terms & conditions


## Complete the Procedure, Maximize Savings

Enjoy **5% extra off** when buying instruments for 2 phases, or **10% extra off** for all 3 phases!

### CARE

Ensuring an optimally targeted non-surgical treatment choice

ASSESS

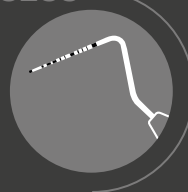


CARE



BOOST




## Micro Mini Five™ Gracey Curettes Harmony™ Handle

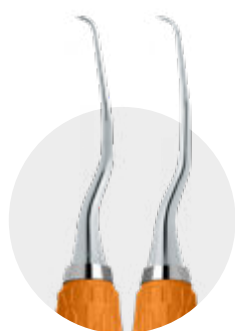

#11/12  
SMS11/12XCE2

79,50 €  
**67,60 €**

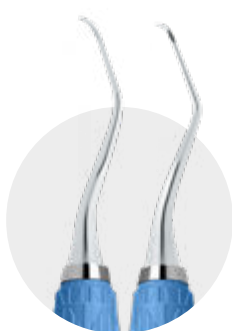

#13/14  
SMS13/14XCE2

79,50 €  
**67,60 €**



#7/8  
SMS7/8XCE2

79,50 €  
**67,60 €**



#1/2  
SMS1/2XCE2

79,50 €  
**67,60 €**

## BioGent Labriola-Cortellini Gracey Curettes #9 Handle



#1/2  
SG1/29LCE2

66,50 €  
**56,50 €**



#7/8  
SG7/89LCE2

66,50 €  
**56,50 €**



#11/12  
SG11/129LCE2

66,50 €  
**56,50 €**



#13/14  
SG13/149LCE2

66,50 €  
**56,50 €**


## Complete the Procedure, Maximize Savings

Enjoy **5% extra off** when buying instruments for 2 phases, or **10% extra off** for all 3 phases!

### CARE

Ensuring an optimally targeted non-surgical treatment choice


ASSESS

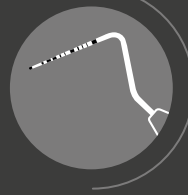

CARE



BOOST

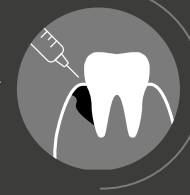

# Piezo Scaling Tips

**Why Piezo?**  
The smooth linear movement generated by piezoelectric systems creates a stroke pattern that is similar to hand instrumentation. Adapting the lateral surface of the piezo tip against the tooth surface (similar to a hand instrument) will increase clinical efficiency.

**S-Series Tips**  
Compatible with Satelec/ Acteon® and NSK® devices\*


**E-Series Tips**  
Compatible with EMS® handpieces\*

**-15%**



Discover more

## Scaling

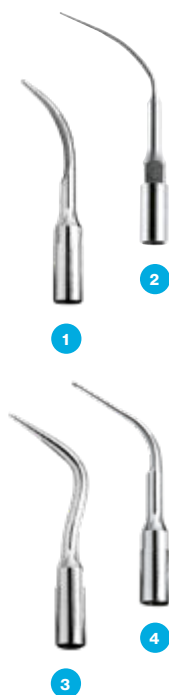

### Description

- S10 Universal**  
Standard diameter tip used to remove heavy supragingival deposits.
- P Universal**  
Standard diameter tip used to remove moderate to heavy deposits located supragingivally and in shallow pockets.
- P10 Universal**  
Standard diameter tip used to remove light deposits and plaque. Tip features smooth edges for safe removal of soft and hard deposits.
- 100 Universal**  
Probe-sized tip used to remove light deposits in shallow to moderate pockets and for periodontal maintenance.

Tip Type	Code	Retail each	Promo
S-Series	USS10	110,00 €	<b>93,50 €</b>
E-Series	UES10	116,00 €	<b>98,60 €</b>
S-Series	USP	115,00 €	<b>97,75 €</b>
E-Series	UEP	120,00 €	<b>102,00 €</b>
S-Series	USP10	116,00 €	<b>98,60 €</b>
S-Series	US100	115,00 €	<b>97,75 €</b>
E-Series	UE100	125,00 €	<b>106,25 €</b>

## After Five™



These probe sized tips are designed to enhance access into periodontal pockets and improve adaptation in interproximal areas, allowing the clinician to more thoroughly debride difficult to reach areas to achieve better clinical results.

Description	Tip Type	Code	Retail each	Promo
<b>1 Straight</b>	S-Series	USAFS	118,00 €	<b>100,30 €</b>
<b>2 Right</b>	S-Series	USAFR	118,00 €	<b>100,30 €</b>
<b>3 Left</b>	S-Series	US AFL	118,00 €	<b>100,30 €</b>

\*Our offered items are compatible with but not original products of device manufacturing companies Acteon, EMS, NSK and/or Satelec and are not produced, licensed or recommended from them. All used brand names and definitions are registered trademarks and brands of respective owners. They are intended only to show compatibility of items with products of different manufacturers.


## Complete the Procedure, Maximize Savings

Enjoy **5% extra off** when buying instruments for 2 phases, or **10% extra off** for all 3 phases!

## CARE

Ensuring an optimally targeted non-surgical treatment choice

ASSESS

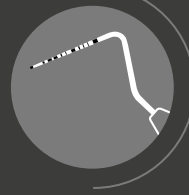


CARE



BOOST



## Perio

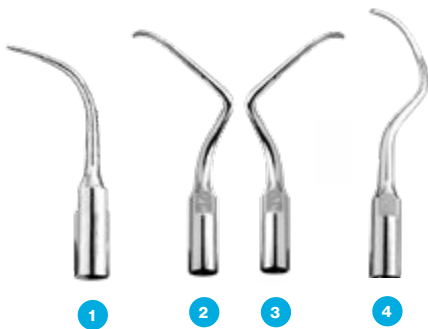

### Description

- 1 Universal Perio** Thin tip used to remove light deposits and plaque. Tip features smooth edges for safe removal of soft and hard deposits.
- 2 Left Perio** Standard diameter contra-angle tip used to remove calculus on all surfaces of premolars and molars, especially difficult to reach interproximal surfaces.
- 3 Right Perio** Standard diameter contra-angle tip used to remove calculus on all surfaces of premolars and molars, especially difficult to reach interproximal surfaces.
- 4 3 Perio** Thin-bladed curette-like tip used to remove heavy deposits and necrotic cementum on anteriors and premolars.

Tip Type	Code	Retail each	Promo
S-Series	USU	110,00 €	<b>93,50 €</b>
S-Series	US4L	135,00 €	<b>114,75 €</b>
S-Series	US4R	135,00 €	<b>114,75 €</b>
S-Series	USP3	120,00 €	<b>102,00 €</b>

## Diamond-Coated



### Description

- 1 Left Diamond** Thin, contra-angle diamond-coated finishing instrument used for removal of residual debris and designed to enhance access to deep periodontal pockets and furcations.
- 2 Right Diamond** Thin, contra-angle diamond-coated finishing instrument used for removal of residual debris and designed to enhance access to deep periodontal pockets and furcations.
- 3 Standard Diamond** Thin, straight diamond-coated finishing instrument used for removal of residual debris and designed to enhance access to deep periodontal pockets primarily in anterior areas.

Tip Type	Code	Retail each	Promo
S-Series	US2LD	110,00 €	<b>106,25 €</b>
E-Series	UE2LD	116,00 €	<b>109,65 €</b>
S-Series	US2RD	115,00 €	<b>106,25 €</b>
E-Series	UE2RD	120,00 €	<b>109,65 €</b>
S-Series	US1ID	115,00 €	<b>97,75 €</b>
E-Series	UE1D	125,00 €	<b>109,65 €</b>

## Which Tip Should I Use?

Application Chart	S10	U	100	P10	P3	4R 4L	1D	2RD 2LD	P	AFS, AFL, AFR
Light to moderate supragingival deposits	●	●	●		●				●	
Heavy supragingival deposits	●			●					●	
Light to moderate deposits in shallow pockets		●	●		●		●		●	●
Light to moderate deposits in deep periodontal pockets					●		●	●		●
Moderate to heavy deposits in deep periodontal pockets						●				
Furcation areas								●		●


## Complete the Procedure, Maximize Savings

Enjoy **5% extra off** when buying instruments for 2 phases, or **10% extra off** for all 3 phases!

### CARE

Ensuring an optimally targeted non-surgical treatment choice


ASSESS

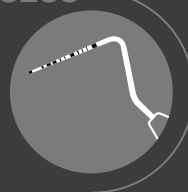

CARE



BOOST

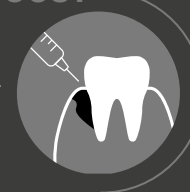

# Magneto Inserts

**-15%**

**Powerful Performance** Provides the strength dentists need to treat a wide range of patients supragingival and subgingival scaling with dedicated scaling, After Five™, Perio, and diamond-coated finishing tip options.

**Enhanced Comfort** Designed to make procedures more comfortable.

**Versatile Options** Multiple tip designs for every clinical preference.



Discover more

## #10 Universal



Tip Description	Recomm.	Usage Tips	Handle Types	Water Delivery	Grip	Part Codes <sup>1</sup>	Retail each	Promo
Standard diameter shank extending to a tapered tip.	<ul style="list-style-type: none"> <li>Light to Heavy Deposits</li> <li>Tenacious Calculus</li> <li>Orthodontic Cement</li> </ul>	Great all purpose insert for supragingival moderate to heavy debris removal.  Supragingival use, can be used subgingival if tissue allows	Swivel Direct Flow	Direct Flow	Silicone	<b>UI30SD10</b>	180,00 €	<b>153,00 €</b>
			Swivel	Base Flow	Silicone	<b>UI25SD10</b>	180,00 €	<b>153,00 €</b>
Tip cross section:			Swivel	Base Flow	Silicone	<b>UI30SS10</b>	170,00 €	<b>144,50 €</b>
						<b>UI25SS10</b>	180,00 €	<b>153,00 €</b>
Tip cross section:			Streamline Direct Flow	Direct Flow	Resin	<b>UI30SF10</b>	165,00 €	<b>140,25 €</b>
						<b>UI25SF10</b>	165,00 €	<b>140,25 €</b>

## #100 Thin



Tip Description	Recomm.	Usage Tips	Handle Types	Water Delivery	Grip	Part Codes <sup>1</sup>	Retail each	Promo
Long thin shank extending to a thin tapered tip.	<ul style="list-style-type: none"> <li>Light to Moderate Deposits</li> </ul>	Excellent choice for a variety of periodontal conditions.  Thin shank allows effective subgingival and interproximal access.	Swivel Direct Flow	Direct Flow	Silicone	<b>UI30SD100</b>	180,00 €	<b>153,00 €</b>
			Swivel	Base Flow	Silicone	<b>UI25SD100</b>	180,00 €	<b>153,00 €</b>
Tip cross section:		Ideal for periodontal maintenance and localized periodontal defects.	Swivel	Base Flow	Silicone	<b>UI30SS100</b>	170,00 €	<b>144,50 €</b>
						<b>UI25SS100</b>	180,00 €	<b>153,00 €</b>
Tip cross section:			Streamline Direct Flow	Direct Flow	Resin	<b>UI30SF100</b>	165,00 €	<b>140,25 €</b>

<sup>1</sup> Inserts designed for 30 kHz units have "30" in the part code. Inserts designed for 25 kHz units have "25" in the part code.


## Complete the Procedure, Maximize Savings

Enjoy **5% extra off** when buying instruments for 2 phases, or **10% extra off** for all 3 phases!

## CARE

Ensuring an optimally targeted non-surgical treatment choice

ASSESS

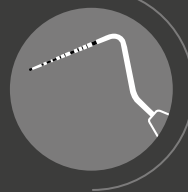

CARE



BOOST



## #1000



Tip Description	Recomm.	Usage Tips	Handle Types	Water Delivery	Grip	Part Codes <sup>1</sup>	Retail each	Promo
Triple bend shank extending to a tapered tip with beveled edges.	<ul style="list-style-type: none"> <li>Moderate to Heavy Deposits</li> <li>Tenacious deposits</li> <li>Orthodontic Cement</li> </ul>	Triple bend shank allows increased access around line angles and interproximally.  Beveled tip edges concentrate energy on corners and enhance removal of heavy calculus and stain.  Supragingival use, subgingival if tissue allows	Swivel Direct Flow	Direct Flow	Silicone	<b>UI30SD1000</b>	180,00 €	<b>153,00 €</b>
			Swivel	Base Flow	Silicone	<b>UI25SD1000</b>	180,00 €	<b>153,00 €</b>
Tip cross section:			Streamline Direct Flow	Direct Flow	Resin	<b>UI30SS1000</b>	170,00 €	<b>144,50 €</b>
						<b>UI25SS1000</b>	180,00 €	<b>153,00 €</b>
						<b>UI30SF1000</b>	165,00 €	<b>140,25 €</b>

## XT



Tip Description	Recomm.	Usage Tips	Handle Types	Water Delivery	Grip	Part Codes <sup>1</sup>	Retail each	Promo
Long, ultra thin shank tapers to a very thin tip.	<ul style="list-style-type: none"> <li>Light to Moderate Deposits</li> <li>Tenacious subgingival deposits</li> </ul>	Site specific uses including narrow, deep pockets, furcations, developmental concavities and tight areas between misaligned teeth.	Swivel	Base Flow	Silicone	<b>UI30SSXT</b>	180,00 €	<b>153,00 €</b>
						<b>UI25SSXT</b>	180,00 €	<b>153,00 €</b>
Tip cross section:			Streamline Direct Flow	Direct Flow	Resin	<b>UI30SFXT</b>	165,00 €	<b>140,25 €</b>

1. Inserts designed for 30 kHz units have "30" in the part code. Inserts designed for 25 kHz units have "25" in the part code.


## Complete the Procedure, Maximize Savings

Enjoy **5% extra off** when buying instruments for 2 phases, or **10% extra off** for all 3 phases!

### CARE

Ensuring an optimally targeted non-surgical treatment choice

ASSESS

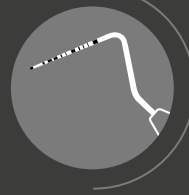

CARE




BOOST



## XT Triple Bend



Tip Description	Recomm.	Usage Tips	Handle Types	Water Delivery	Grip	Part Codes <sup>1</sup>	Retail each	Promo
Thin triple bend shank extending to a thin beveled edge tip 24% thinner than #1000 insert	• Light to Moderate Deposits	Thin triple bend shank allows increased access around line angles and interproximally. Beveled tip concentrates energy at corners allowing for enhanced calculus and stain removal. Supra and subgingival use.	Swivel Direct Flow	Direct Flow	Silicone	<b>UI30SDXTTB</b>	180,00 €	<b>153,00 €</b>
Tip cross section: 			Streamline Direct Flow	Direct Flow	Resin	<b>UI30SFXTTB</b>	165,00 €	<b>140,25 €</b>

## P10 & P10P



Tip Description	Recomm.	Usage Tips	Handle Types	Water Delivery	Grip	Part Codes <sup>1</sup>	Retail each	Promo
Standard diameter shank extending to a thin tapered tip. Original prophy style inserts with external water tube.	• Light to Heavy deposits • Orthodontic cement	Supra gingival and subgingival	Original Prophy Style	External tube	Metal	<b>UI30KP10</b> <b>UI25KP10</b>	155,00 € 145,00 €	<b>131,75 €</b> <b>123,25 €</b>
Tip cross section: 			Prophy Plus	External tube	Resin	<b>UI30KP10P</b> <b>UI25KP10P</b>	160,00 € 155,00 €	<b>136,00 €</b> <b>131,75 €</b>

<sup>1</sup> Inserts designed for 30 kHz units have "30" in the part code. Inserts designed for 25 kHz units have "25" in the part code.


## Complete the Procedure, Maximize Savings

Enjoy **5% extra off** when buying instruments for 2 phases, or **10% extra off** for all 3 phases!

## CARE

Ensuring an optimally targeted non-surgical treatment choice


ASSESS

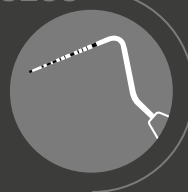

CARE




BOOST

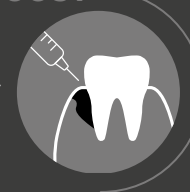


## After Five Left



Tip Description	Recomm.	Usage Tips	Handle Types	Water Delivery	Grip	Part Codes <sup>1</sup>	Retail each	Promo
Extended thin shank narrows to a tapered, probe sized tip.	• Light to moderate deposits	Allows improved access to posterior proximal surfaces. Can be used in furcations. Available alone or as a set with right and straight inserts.	Swivel Direct Flow	Direct Flow	Silicone	<b>UI30SDL</b> <b>UI25SDL</b>	180,00 €	<b>153,00 €</b>
			Streamline Direct Flow	Direct Flow	Resin	<b>UI30SFL</b>	165,00 €	<b>140,25 €</b>
Tip cross section: 			After Five Plus	External water tube	Resin	<b>UI30KSF10L</b> <b>UI25KSF10L</b>	160,00 €	<b>136,00 €</b>
			After Five	External water tube	Metal	<b>UI30KSL10L</b> <b>UI25KSL10L</b>	155,00 €	<b>131,75 €</b>


## After Five Right



Tip Description	Recomm.	Usage Tips	Handle Types	Water Delivery	Grip	Part Codes <sup>1</sup>	Retail each	Promo
Extended thin shank narrows to a tapered, probe sized tip.	• Light to moderate deposits	Allows improved access to posterior proximal surfaces. Can be used in furcations. Available alone or as a set with left and straight inserts.	Swivel Direct Flow	Direct Flow	Silicone	<b>UI30SDR</b> <b>UI25SDR</b>	180,00 €	<b>153,00 €</b>
			Streamline Direct Flow	Direct Flow	Resin	<b>UI30SFR</b> <b>UI25SFR</b>	165,00 €	<b>140,25 €</b>
Tip cross section: 			After Five Plus	External water tube	Resin	<b>UI30KSF10R</b> <b>UI25KSF10R</b>	160,00 €	<b>136,00 €</b>
			After Five	External water tube	Metal	<b>UI30KSL10R</b> <b>UI25KSL10R</b>	155,00 €	<b>131,75 €</b>

## After Five Straight



Tip Description	Recomm.	Usage Tips	Handle Types	Water Delivery	Grip	Part Codes <sup>1</sup>	Retail each	Promo
Extended thin shank narrows to a tapered, probe sized tip. 40% thinner than original proph design.	• Light to moderate deposits	Used to remove calculus from periodontal pockets with improved adaptation for deeper pockets. Available alone or as a set with left and right inserts.	Swivel Direct Flow	Direct Flow	Silicone	<b>UI30SDS</b> <b>UI25SDS</b>	180,00 €	<b>153,00 €</b>
			After Five Plus	External water tube	Resin	<b>UI30KSF10S</b> <b>UI25KSF10S</b>	160,00 €	<b>136,00 €</b>
Tip cross section: 			After Five	External water tube	Metal	<b>UI25KSL10S</b>	155,00 €	<b>131,75 €</b>

<sup>1</sup> Inserts designed for 30 kHz units have "30" in the part code. Inserts designed for 25 kHz units have "25" in the part code.


## Complete the Procedure, Maximize Savings

Enjoy **5% extra off** when buying instruments for 2 phases, or **10% extra off** for all 3 phases!

### BOOST

Leveraging the physiological healing capacity

ASSESS

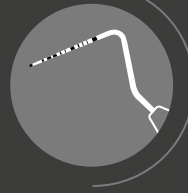

CARE



BOOST



# Sterify Gel

## Supporting the Tissue Healing.

**-15%**



**Sterify Gel - 0.3 ml, 1 syringe**  
**STY-GEL0301**

59,00 €

**50,10 €**

Meet the **reduced bacterial contamination solution to support the healing process** in the treatment of periodontal disease.

**Sterify Gel** is a medical device that offers a new approach to periodontal care. Mucoadhesive polymeric hydrogel in a pre-filled syringe for the treatment of periodontal disease<sup>2</sup>.

### Sterify Gel THE MAIN BENEFITS:



**Reduction of bacterial contamination** thanks to its occluding action



**Support to tissue healing** with significant and proven constant improvement of pocket depth



**No antibiotics, no antiseptics and no chlorhexidine** but simultaneously acts as broad-spectrum antibacterial barrier, without generating resistance



**No animal-derived components**



**Stable at room temperature** with no need to manage the cold chain



**Easy to use**



[Discover more](#)


# Targeted Healing Approach Starter Packs



**With these Starter Packs Promotions  
you will combine clinical efficiency  
and economic benefits for your  
practice.**


# Targeted Healing Approach Starter Packs

**ASSESS** > **CARE** > **BOOST**




**1x EFP Probe**  
**+ 1x Sterify Gel**

101,40€ **59 €**




**ASSESS** > **CARE** > **BOOST**



**4x Harmony™ Currettes of your choice**  
**+ 1x Harmony™ Scaler of your choice**  
**+ 1x EFP Probe**  
**+ 1x Sterify Gel**

419,00€ **229 €**



COMPLETE SOLUTION

**ASSESS** > **CARE** > **BOOST**



**4x Sterify Gel**

236,00€ **177 €**



# Targeted Healing Approach Starter Packs

ASSESS CARE BOOST

**Piezo Tips**

**2+1**  
VALID FOR  
THE WHOLE  
RANGE

**THE CHEAPEST FOR FREE**



The image shows a close-up of a piezo dental handpiece tip. The tip is metallic and has a blue ring near the handle. It is emitting a fine, white spray of particles, likely water or air, which is directed towards the left. The background is dark, making the spray stand out.

ASSESS CARE BOOST

**Magneto Tips**

**3+1**  
VALID FOR  
THE WHOLE  
RANGE


**THE CHEAPEST FOR FREE**



The image displays a row of ten magneto dental handpiece tips. Each tip has a different colored handle: blue, red, teal, green, silver, purple, dark purple, orange, grey, and pink. The tips are arranged in a slightly overlapping, diagonal line from the bottom left to the top right. The background is dark.

# Targeted Healing Approach Starter Packs

ASSESS

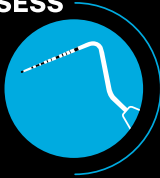

CARE



BOOST



1x EFP Probe  
+ 1x Harmony™ Scaler of your choice  
+ 1x Piezo Tip of your choice  
+ 1x Sterify Gel


315,40 € **236 €**



**COMPLETE SOLUTION**

# Targeted Healing Approach Starter Packs

ASSESS

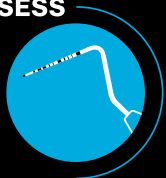

CARE



BOOST



1x EFP Probe  
+ 1x Harmony™ Scaler of your choice  
+ 1x Magneto Tip of your choice  
+ 1x Sterify Gel

360,40 € **281 €**



**COMPLETE SOLUTION**

# Preparation



**Prepare and protect your chairside at best for the Periodontal Treatment with Omnia non-sterile disposable products.**

**In the dental practice, the prevention of cross-contamination is an absolute priority. For this reason, Omnia offers a wide range of non-surgical disposable products, designed to ensure maximum safety in your daily practice, for every procedure where sterility is not required, such as oral hygiene, endodontics, restorative dentistry, and orthodontics.**

Our range of disposable protection is designed to protect clinicians, assistants and patients from possible exposure to pathogens. Each product makes dental practices easier in perfect safety.

## Non-Sterile Disposable Products

-15%



### OmniKit 100 Kit of Non-Sterile Disposable Products Containing Supplies for 100 Patients

The kit contains:

- 100 two-ply napkins, 45 x 33 cm
- 100 saliva ejectors, 13 cm, with removable cap
- 100 disposable plastic cups
- 60 bibs on a roll, 50 x 80 cm

Blue **30.A1000.00**

Lilac **30.A1002.00**

Fresh green **30.A1001.00**

62,00 €

**52,70 €** Available only in Italy and Germany.



### Headrest Covers

To isolate the headrest of the dental chair, these waterproof, absorbent, comfortable, and versatile non-woven fabric (NWF) protections provide not only an effective barrier but also give your dental unit a clean and pleasant appearance, offering your patient a sense of safety and reliability.

### Omnia Headrest cover - White **20.01300.00**

28,60 €

**24,31 €**



Discover  
more

## Non-Sterile Disposable Products

**-15%**

### Seat and Backrest Covers

To ensure hygienic conditions for medical staff and patients, these protective covers provide an effective barrier for the seat and backrest of the dental chair, as well as for the operator's stool. The universally sized covers are made of waterproof non-woven fabric (NWF) and feature ties and elastic bands for secure fitting.

#### Omnia - Universal-size seat/back covers - White

**20.00041.00**~~62,70 €~~**53,30 €**

# Sharpening



**You rely on your dental instruments to be an extension of your skills, and keeping them in optimal condition will help you perform at your best. Sharp instruments improve deposit removal, increase tactile sensitivity, and allow for greater instrument control.\***

**No matter what your style of sharpening is, HuFriedyGroup can help you achieve your best clinical outcomes with our Total Sharpening Solution.**

## **CHOOSE YOUR STYLE:**

- **CLASSIC** The original sharpening technique: instrument in one hand, your favorite sharpening stone in the other. HuFriedyGroup's extensive selection of stones and cards provides many options to meet the needs and preferences for this classic style of sharpening.
- **CONTEMPORARY** For the clinician who prefers an innovative approach to sharpening, HuFriedyGroup's Sidekick™ is the answer. This portable and easy-to-use sharpener will provide consistent results every time.

\* Fundamentals of Periodontal Instrumentation & Advanced Root Instrumentation, J. Nield-Gehrig, 6th Edition, Lippincott Williams & Wilkins, 2008

Sharpening

-15%



**CLASSIC - Diamond Sharpening Cards Set**

Includes 1 each: Extra Fine, Fine and Medium Grid Diamond Sharpening Card.

**DSCSET**

249,50 €  
**212,10 €**



**CONTEMPORARY - SideKick™ Sharpener Kit**

Kit includes: Sidekick Unit, User Manual, Ceramic Sharpening Stone, Plastic Test Stick, Magnifying Lens and 2 AA Batteries.

**SDKKIT**

565,50 €  
**480,70 €**





Discover more

## Who We Are

We are a combination of companies that have come together under one roof to deliver the products, services, and solutions you need to be the best in practice. With our passion for infection prevention solutions, premium instrumentation, and service and support programs, HuFriedyGroup will help you perfect your craft and continue to deliver the highest level of care to each patient.



## Our Contacts

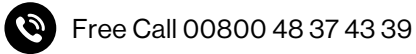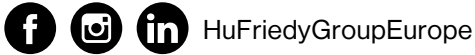

Hu-Friedy Mfg. Co., LLC • European Headquarters • Lyoner Str. 9 • 60528 Frankfurt am Main, Germany • HuFriedyGroup.eu

As some of the products included in this catalogue might not be available in some countries, we invite you to verify the availability of the product with your local authorised dealer or to contact our customer care at the free phone number 00800 48 37 43 39. Available while stocks last.

All company and product names are trademarks of Hu-Friedy Mfg. Co. LLC, its affiliates or related companies, unless otherwise noted.

©2026 Hu-Friedy Mfg. Co., LLC All rights reserved. HFL-1002GB/0526